

# How to use UniGuard®


## Single use pen needle remover

Follow the steps below when using **UniGuard®** to remove pen needles.



1. Hold the **UniGuard®** between the thumb and forefinger of one hand. Then push the pen down firmly until the pen clicks into place on the cradle.

**Do not push the pen into the UniGuard® tip first.**

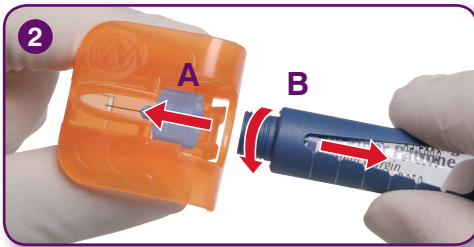

2. **A.** Push the pen forwards until it clicks again.  
**B.** Squeeze the **UniGuard®** tightly, unscrew the pen needle and pull the pen away.



3. Push the safety flap up firmly until it snaps shut with an audible click.
4. Dispose of the **UniGuard®** in a sharps bin in accordance with local regulations.



Pack Size	50
OM code	AN0682



SINGLE USE