


Hemo_Family

Cleaning of devices of the Hemo_Family

EKF
DIAGNOSTIC



1 At least once per month the optical unit must be cleaned. If the displayed value of the control cuvette differs from the value shown on the label or if the error message arises "intensity too low" the optical unit must be cleaned as well.

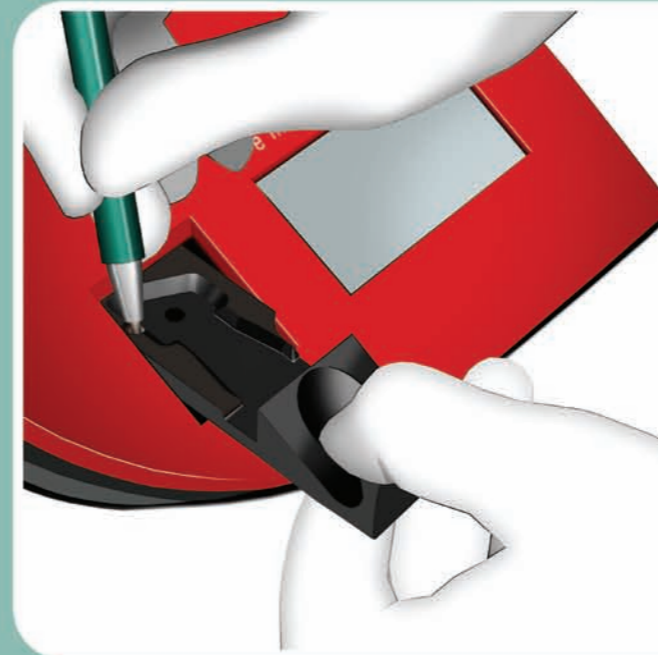

2 Open the cuvette holder until a noticeable resistance. Press down the latch on the left-hand side of the cuvette holder with the help of a pointy object. Pull out the cuvette holder forward at the same time.



3 Insert the cleaner carefully into the opening of the cuvette holder until you can feel a smooth resistance. Slightly push the cleaner down and then deeper into the opening.



4 Don't use any force, only a slight resistance should be felt. Wipe the optical unit several times in order to clean it. Remove the cleaner. If the used cleaner is soiled repeat the cleaning process with a new one.



5 The cuvette holder will be cleaned with a mild soap solution. For disinfection, standard solvent-free products can be used similarly as for surface disinfection.



6 Wait at least 15 min until the cuvette holder is dry and re-insert it. In order to replace the cuvette holder simply push it into the opening of the housing in the correct position until it is locked in place.



7 Cleaning the housing and touch screen is best accomplished with a lint-free cloth lightly dampened with clear water, a mild soap solution or standard solvent-free products similarly as for surface disinfection.

EKF-diagnostic GmbH
Ebendorfer Chaussee 3
39179 Barleben
Germany

Tel.: +49-(0)39203-785-0
Fax: +49-(0)39203-785-16
info@EKF-diagnostic.com
www.EKF-diagnostic.com

Version 3.0-06/2010
Subject to technical modifications
3000-6245-0864

HEMO-FAMILY

The rapid
Hb-Test